Pacific Hemostasis® Calcium Chloride Solution (0.02M)

I. Intended Use

Pacific Hemostasis Calcium Chloride Solution 0.02M (CaCl₂) is intended for use with ellagic acid activators in performing the activated partial thromboplastin time (use with APTT-LS and APTT-XL). Refer to product numbers 100402 and 100403 (APTT-XL) Refer to product numbers 100300 and 100303 (APTT-LS)

II. Reagent

For *in vitro* diagnostic use.

Composition: 0.222% calcium chloride, 0.1% sodium azide and stabilizers.

Warning: Calcium Chloride Solution (0.02M) contains sodium azide. Sodium azide under acid conditions yields hydrazoic acid, an extremely toxic compound. Dilute with running water before discarding, and then flush with large volumes of water. These precautions are recommended to avoid deposits in metal piping in which explosive conditions may develop.

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FISHER DIAGNOSTICS® LIMITED WARRANTY

Fisher Diagnostics (FD) warrants to the purchaser only that FD products will perform as described on their labeling and product literature. Purchaser must determine the suitability of FD products for their specific applications. FD's sole obligation will be, at its option, to either replace a non-conforming or defective product, or return the purchase price. FD DISCLAIMS ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTIES OF MERCHANTABILITY AND FITNESS FOR ANY PARTICULAR PURPOSE. Neither FD nor its affiliates shall, in any event, be liable for incidental or consequential loss or damage.

All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Bestellinformationen			
Kat.-Nr.	Beschreibung	Inhalt	
100304	CaCl ₂ (0,02 M)	10 x 10 mL	
100300	APTT-LS	10 x 4 mL	
100303	APTT-LS	10 x 10 mL	
100402	APTT-XL	10 x 4 mL	
100403	APTT-XL	10 x 10 mL	

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All other trademarks are the property of Thermo Fisher Scientific Inc. and its subsidiaries.

Pacific Hemostasis Calcium Chloride Solution (0.02M)

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