
Shandon PapSpin™

Protocol for Obtaining Samples For Gynaecological Specimens

- 1 Label PapSpin Collection Vial with appropriate patient identification.
- 2 Remove the cap from the PapSpin Collection Fluid Vial and leave the open vial on a table near you.
- 3 Insert the speculum **without the use of a lubricant**.
- 4 Contact the cervix with the PAPETTE™ brush and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix. (See diagram).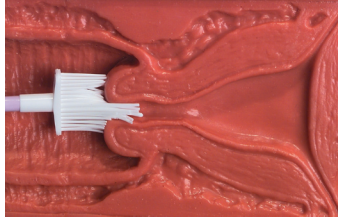

- 5 While maintaining gentle pressure in the direction of the cervix, rotate the brush 5 times in a clockwise direction. Remove the PAPETTE™ brush.

DO NOT PUT THE PAPETTE™ BRUSH DOWN.

- 6 Use forceps or gloved fingers to pull the white bristle end of the PAPETTE™ brush from its handle and drop it into the vial of PapSpin Collection Fluid.

DO NOT DELAY THIS STEP FOR ANY REASON.

- 7 Replace the lid onto the vial and make sure it is tightly secured. Shake the vial and prepare it for transport.

Handling and Storage of PapSpin Collection Fluid

Handling: Keep containers closed when not in use. Wear protective gloves and splash-proof goggles when handling this solution. An impermeable apron is also recommended as a prudent measure of protection. Avoid all contact with skin and eyes. Do not continue to wear contaminated clothing after a spill.

Storage: Store product in a non-flammable storage cabinet. Keep away from heat, sparks and naked flames.

Thermo

ELECTRON CORPORATION

Anatomical Pathology
Clinical Diagnostics
171 Industry Drive
Pittsburgh
PA 15275, USA

Tel: 1-800-547-7429
+1 412 788 1133
Fax: +1 412 788 1138
www.thermo.com/shandon

**To re-order Shandon
Papspin, order part
number A78710006 from
your local supplier.**

Anatomical Pathology
Clinical Diagnostics
93-96 Chadwick Road
Astmoor, Runcorn
Cheshire, WA7 1PR, UK

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