Laboratory Protocol For Preparation Of Cervical Cytological Samples Using The Shandon PapSpin™ Test Method with Shandon Cytocheck

(Note - the numbered steps in this document refer to label instruction illustrations)

Specimen Inspection and Slide Preparation

(see A78310258 for details about obtaining sample)

SPECIMENS IN SHANDON PAPSPIN COLLECTION FLUID MUST BE PROCESSED WITHIN 30 DAYS OF COLLECTION

- 1 The cervical collection broom head must be covered by preservative. Reject specimens that do not have the broom head in the vial. Note any deviations according to laboratory procedures.
- 2 Make sure the cap of each vial is secure. Vortex each Shandon PapSpin vial for 10 seconds.
- 3 Write the Specimen Identification Number at the top of the frosted label of the Shandon EZ Megafunnel™ Slides. Pre-labelling the slides near the top will simplify the matching of the chamber with the proper sample.

Shandon Cytospin Sealed Head Preparation

- 4 Place the Shandon Cytospin® Sealed Head onto the work surface of a biological safety cabinet and remove the lid. Prepare Shandon EZ Megafunnel assemblies.
- 5 Load the Sealed Head with the assembled Shandon EZ Megafunnels. Make sure that the Shandon EZ Megafunnels are evenly distributed in the Sealed Head to ensure that it is balanced.

Specimen Transfer

- 6 Gently agitate the sample vial to resuspend the cells *(see Shandon CytoCheck Operator Guide (B1004010) for more details)*
- 7-12 Immediately place the vial in the sample holder of the Shandon CytoCheck so that the tip of the collection brush falls within the red area marked on the edge of the sample holder.

Remove the vial from the Shandon CytoCheck sample holder and gently agitate the sample vial to resuspend the cells.

Immediately pipette the displayed amount of cell suspension into the corresponding labelled Shandon EZ Megafunnel assembly, followed by 1.0ml of fresh Shandon PapSpin Collection Fluid¹.

PIPETTE IMMEDIATELY AFTER EACH VORTEXING TO AVOID CELL SETTLING AND NON-UNIFORM SAMPLING. Use of a disposable, graduated plastic bulb pipette is suggested.

- 13 Cap each Shandon EZ Megafunnel.

Replace the lid of the Sealed Head and push down on the centre button to lock. Place the Sealed Head into the Shandon Cytospin and close the Shandon Cytospin lid.

Cytocentrifugation

- 14 Spin at 1,250 rpm for 5 minutes with HIGH acceleration (HI).

Slide Removal

- 15-16 Remove each Shandon EZ Megafunnel assembly individually. Tilt the assembly downward to keep the fluid away from the slide to avoid washing off cells.
- 17 Remove each slide by lifting the slide up and away from the Shandon EZ Megafunnel to avoid smearing the cells.
- 18 Lay the slides flat and allow to air dry completely.
- 19 Discard the Shandon EZ Megafunnel in accordance with standard laboratory practice.

MAKE SURE THAT GOOD LABORATORY PRACTICE IS FOLLOWED WHEN POTENTIALLY HAZARDOUS MATERIALS ARE HANDLED

Processing

Prior to staining the prepared slide, immerse the slide in 95% ethanol for 10 minutes.

Stain the slides.

Coverslip and screen.

- ¹ If the Shandon PapSpin slide is too cellular, increase the dilution in the Shandon Megafunnel with fresh Shandon PapSpin Collection Fluid and prepare another slide.

THE SHANDON EZ MEGAFUNNEL IS SINGLE USE ONLY - Do not use more than once.

Specimen stability: Specimens fixed in PapSpin Collection Fluid are stable for at least 30 days stored at room temperature.

Thermo

ELECTRON CORPORATION

Clinical Diagnostics
Anatomical Pathology
171 Industry Drive
Pittsburgh
PA 15275, USA

Tel: 1-800-547-7429
+1 412 788 1133

Fax: +1 412 788 1138
www.thermo.com/shandon

**To re-order Shandon
Papspin, order part
number A78710006 from
your local supplier.**

Clinical Diagnostics
Anatomical Pathology
93-96 Chadwick Road
Astmoor, Runcorn
Cheshire, WA7 1PR, UK

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