Infinity™ Lithium Standard

INTENDED USE

This standard is intended for the calibration of Lithium assays using the Thermo Infinity Lithium reagent. It is for in vitro diagnostic use only.

REAGENT COMPOSITION

The Lithium Chloride standard is prepared from a stock Lithium solution (Lithium dissolved in HCl) diluted in physiological saline (0,5 g/L).

This standard contains 2.00 mmol/L of Lithium Chloride with preservative.

WARNING: Do not ingest. Avoid contact with skin and eyes. If spilled thoroughly wash affected areas with water. Standard contains sodium azide which may react with copper or lead plumbing. Flush with plenty of water when disposing. For further information consult the Thermo Infinity Lithium Standard Material Safety Data Sheet.

PREPARATION

The standard is ready to use.

STORAGE AND STABILITY

Once opened the standard is stable until the expiry date stated on the label provided the bottle is stored capped and at 2-8°C.

The solution should be clear, colourless and free of precipitate. Any discolouration or turbidity indicates deterioration and the solution should be discarded.

PROCEDURE

Thermo Infinity Lithium Standard should be handled in the same manner as a patient sample following the assay procedure detailed in the Thermo Infinity Lithium reagent package insert.

PERFORMANCE

Each batch of standard is assayed using Thermo Infinity Lithium reagent to confirm the stated value.

LIMITATIONS

This standard has only been validated for use with the Thermo Infinity Lithium Reagent. Users should verify its suitability for use with other Lithium methods.

TRACEABILITY

The standard has been verified against available Standard Reference Materials from the US National Institute of Standards and Technology (NIST).

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© 2008 Thermo Fisher Scientific Inc. All rights reserved.
a division of Fisher Scientific Company, LLC
a subsidiary of Thermo Fisher Scientific Inc.
MDCI Ltd.