# EN Konelab™/ T Series ISE CALIBRATORS

REF	984031 ISE Calibrator 1	4 x 400 ml
	984035 ISE Calibrator 2 & 3	2 x 20 ml + 2 x 20 ml
	984034 ISE Calibrator 4	2 x 20 ml

**THIS PACKAGE INSERT IS APPLICABLE FOR USE OUTSIDE THE US. ANY REFERENCE TO THE KONELAB SYSTEMS ALSO REFERS TO THE T SERIES.**

## INTENDED USE

For the *in vitro* diagnostic use in Konelab analyzers 60i, 30i, 20i and 20XTi.

The ISE Calibrators 1, 2 & 3 and 4 are intended for calibration of ion selective electrodes for quantitative measurements of K<sup>+</sup>, Na<sup>+</sup>, Cl<sup>-</sup>, ionized Ca<sup>2+</sup> and pH in serum or plasma or Li<sup>+</sup> in serum or K<sup>+</sup> and Na<sup>+</sup> in urine.

## COMPOSITION

NaCl, CH<sub>3</sub>COONa, KCl, CaCl<sub>2</sub>, MgCl<sub>2</sub>, LiCl, buffer and preservative.

These products do not contain any human constituents.

## PRECAUTIONS

For *in vitro* diagnostic use only.  
Do not swallow.

## HANDLING

Liquid, ready for use.  
Use 2 ml of calibrator solutions in cups for the calibration.  
Do not store calibrators 2 & 3 and 4 in cups in analyzer more than 1 hour.

It is recommended to do calibration at least once a month and always when electrodes have been replaced or every time a new lot of bottle or bag of ISE Calibrator 1,2,3,4 is used or whenever control results require recalibration.

Note: Check that there are no bubbles in the calibration cups or on the surface of calibrators 2, 3 and 4 when you insert cups into the Konelab analyzer.

## LIMITATIONS

Do not use any bag or vial that has visible evidence of microbial growth or any damage.

## STORAGE AND STABILITY

Reagents in unopened bags or vials are stable at 2...30 °C until the expiration date printed on the label.

After opening ISE Calibrators 2 & 3 and 4 are stable for 18 weeks.

Store ISE Calibrators 2 & 3 and 4 tightly capped when not in use.

After placing ISE Calibrator 1 to analyzer it is usable for 18 weeks.

## ASSIGNED VALUES (1)

Concentrations are given in mmol/l.

Ion	ISE Cal 1	ISE Cal 2	ISE Cal 3	ISE Cal 4
K <sup>+</sup>	4.5	6.0	3.0	150
Na <sup>+</sup>	140	120	150	50
Ca <sup>2+</sup>	1.0	0.5	2.0	
Cl <sup>-</sup>	104	87	110	
Li <sup>+</sup>	1.0	0.5	2.0	
pH	7.38 (37°C)	6.84 (37°C)	7.38 (37°C)	

## TRACEABILITY

Na<sup>+</sup>: NIST SRM 919  
Cl<sup>-</sup>: NIST SRM 909, 956  
K<sup>+</sup>: NIST SRM 918  
Li<sup>+</sup>: NIST SRM 924  
Ca<sup>2+</sup>: JCCRM 321-1

pH measurement system is calibrated using solutions traceable to the NIST pH scale.

## TEST PROCEDURE

More information see Konelab Reference Manual.

## QUALITY CONTROL

Available controls:  
Nortrol, code 981043  
Abtrol, code 981044  
Select Ion Low, code 981059  
Select Ion Normal, code 981058  
Select Ion High, code 981094  
It is recommended to use two level controls.  
Use quality control samples at least once a day and after each calibration and every time a new bottle or bag of calibrators is used.

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**THIS PACKAGE INSERT IS APPLICABLE FOR USE IN THE US ONLY.**

**INTENDED USE**

For *in vitro* diagnostic use on the Konelab 20XTi, Konelab PRIME 60i, T60 analyzers. The ISE Calibrators 1 and 2 & 3 are intended for calibration of ion selective electrodes for quantitative measurements of K<sup>+</sup>, Na<sup>+</sup>, Cl<sup>-</sup> in serum or plasma using methods defined by Thermo Fisher Scientific Oy.

**PRODUCT INFORMATION**

Kit code 984031 contains 4 bags of 400 ml ISE Calibrator 1 and kit code 984035 contains two bottles of 20 ml ISE Calibrator 2 and two bottles of 20 ml ISE Calibrator 3.

**COMPOSITION**

The ISE calibrators contain NaCl, CH<sub>3</sub>COONa, KCl, CaCl<sub>2</sub>, MgCl<sub>2</sub>, LiCl, buffer and preservative(s).  
These products do not contain any human or animal constituents.

**PRECAUTIONS**

For *in vitro* diagnostic use only. Exercise the normal precautions required for handling all laboratory reagents.  
Do not swallow.

**HANDLING**

Liquid, ready for use.  
Use 2 ml of calibrator solutions in sample cups for the calibrating the analyzer.  
Do not store calibrators 2 & 3 in cups in the analyzer for more than 1 hour.  
It is recommended that the analyzer is calibrated at least once per month, when electrodes have been replaced, when a new calibrator is used, or when control results require recalibration.  
Note: Check that there are no bubbles in the calibration cups or on the surface of calibrators 2 and 3 when you insert cups into the analyzer.

**STORAGE AND STABILITY**

Reagents in unopened bags or vials are stable at 2...30 °C until the expiration date printed on the label.  
After opening, ISE Calibrators 2 & 3 are stable for 18 weeks. Store ISE Calibrators 2 & 3 tightly capped when not in use.  
After placing ISE Calibrator 1 in the analyzer, it is usable for 18 weeks.  
Do not freeze the calibrators.

**TEST PROCEDURE**

For more information, see Reference Manual.

**Quality Control**

Available controls:  
Nortrol, kit code 981043  
Abtrol, kit code 981044  
It is recommended to use at least two levels (low and high) of controls or sample pools.  
Use quality control samples at least once per day and after each calibration and every time a new calibrator is used.

**LIMITATIONS**

Do not use any bag or vial that demonstrates visible evidence of microbial growth or damage.  
For diagnostic purposes, the results should always be assessed in conjunction with the patient's medical history, clinical examination and/or other findings.

**ASSIGNED VALUES**

Concentrations are given in mmol/l (1)

Ion	ISE Cal 1	ISE Cal 2	ISE Cal 3
K <sup>+</sup>	4.5	6.0	3.0
Na <sup>+</sup>	140	120	150
Cl <sup>-</sup>	104	87	110

**Conversion Factor**

K<sup>+</sup>  
mmol/l x 1 → mEq/l  
Na<sup>+</sup>  
mmol/l x 1 → mEq/l  
Cl<sup>-</sup>  
mmol/l x 1 → mEq/l

**Traceability**

Na<sup>+</sup> NIST SRM 919  
Cl<sup>-</sup> NIST SRM 909, 956  
K<sup>+</sup> NIST SRM 918

**BIBLIOGRAPHY**

1. Data on file at Thermo Fisher Scientific Oy.

**MANUFACTURED BY**

Thermo Fisher Scientific Oy  
Clinical Diagnostics Finland  
Ratastie 2, P.O. Box 100, FI-01621 Vantaa, Finland  
Tél. +358 9 329 100, Fax +358 9 3291 0300  
www.thermoscientific.com

**Date of revision (yyyy-mm-dd)**  
2011-08-23

**Changes from previous version**

The line beside the text indicates changes.

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1. Data på fil hos Thermo Fisher Scientific Oy.