EN
HbA1c Control Abnormal

REF 981696 5 x 1 ml

THIS PACKAGE INSERT IS APPLICABLE FOR USE OUTSIDE THE US.**INTENDED USE**

HbA1c Control Abnormal is intended for use on Konelab and Indiko Clinical Chemistry Analyzers as a precision control sample for the HbA1c test.
Any reference to the Konelab systems also refers to the T Series.

COMPOSITION

HbA1c Control Abnormal is a lyophilized, hemolyzed control based on human and sheep's blood. Reactive components are Hb and HbA1c.

PRECAUTIONS

For *in vitro* diagnostic use only.
This product has been tested, and found negative for HBsAg and for antibodies to HIV 1/2- and HCV. However, the product must be handled just as carefully as the patient samples. Bottles contain TTAB (Tetracycltrimethylammonium bromide) 7.7%. Irritating to eyes and skin. Avoid contact with skin and eyes. Flush affected areas with copious amounts of water. See separate sheet for Risk- and Safety Phrases (R36/38;S24/25).

HANDLING

To reconstitute HbA1c Control Abnormal,
1) Remove the cap and carefully free the rubber stopper.
2) Add exactly 1.00 ml of distilled or deionized water.
3) Close the vial and allow to stand for 30 minutes.
4) Dissolve the contents by swirling gently, avoiding the formation of foam.

Note: HbA1c Control Abnormal does not require pretreatment with the hemolyzing reagent.

Let the controls be a maximum of one hour on the analyzer.

STORAGE AND STABILITY

HbA1c Control Abnormal is stable as follows:

In an unopened vial	Until the expiration date when stored at 2...8 °C
After reconstitution	8 hours at 15...25 °C 7 days at 2...8 °C 3 months at -20 °C*

* Freeze only up to a maximum of 4 times in the original bottle.

Store controls tightly capped when not in use.

LIMITATIONS

Do not use any vial that has visible evidence of microbial growth.

ASSIGNED VALUES AND RANGES

The concentrations of the components are lot-specific. Control values are listed in the analyzer specific value sheets.

Assigned values for the components of HbA1c Control Abnormal are given for the methods defined by Thermo Fisher Scientific Oy. The range is the assigned value ± 3 sd. It is recommended that every laboratory establish its own means and acceptable ranges. The control intervals and limits must be adapted to the individual laboratory requirements.

CHANGING CONTROL VALUES

Check always the lot number of the control in your hand. Note that an additional letter, e.g. A, at the end of numeric lot code specifies the packaging time.

Konelab

If the lot number is new, change the respective values in the parameter list manually.

Indiko

If the lot number is new, you can read the respective values either from a barcode or from a file, or you can enter the values manually. For more information, see the *Operation Manual* for Indiko.

MANUFACTURED BY

Thermo Fisher Scientific Oy
Clinical Diagnostics Finland
Ratastie 2, P.O. Box 100, FI-01621 Vantaa, Finland
Tel. +358 9 329 100, Fax +358 9 3291 0300
www.thermoscientific.com

Date of revision (yyyy-mm-dd)

2011-09-22

Changes from previous version

The line beside the text indicates changes.

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