EN
Konelab™/ T Serie
ISE Prime Serum

REF 981990 10 x 5 ml

**THIS PACKAGE INSERT IS
APPLICABLE FOR KONELAB /
T SERIES OUTSIDE THE US.**

INTENDED USE

Priming solution for the ISE (ion selective electrode) unit on Konelab/T Series clinical chemistry analyzers.

PRODUCT INFORMATION

Kit code 981990 contains 10 vials of 5 ml ISE Prime Serum product.

COMPOSITION

ISE Prime Serum is prepared from human serum. This product also contains non-human constituents. Bacteriostatic agents have been added.

PRECAUTIONS

For the priming of the ISE unit on Konelab/T Series analyzers only. Exercise the normal precautions required for handling all laboratory reagents.

This product has been tested by FDA approved methods, and found negative for HBsAg and for antibodies to HIV 1/2- and HCV. However, the product must be handled just as carefully as the patient samples.

HANDLING

To reconstitute remove the cap and carefully free the rubber stopper avoiding the loss of lyophilizate. Add 5 ml of distilled or deionized water. Close the vial and allow to stand for 30 min at room temperature. Dissolve the contents completely by swirling gently. Do not shake.

STORAGE AND STABILITY

Stable until the expiry date when stored at 2...8°C. After reconstitution ISE Prime Serum is stable for 7 days at 2...8°C and for 30 days at -20°C.

LIMITATIONS

Do not use any vial that has visible evidence of microbial growth.

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Il s'agit d'une solution d'amorçage pour le module ionogramme des analyseurs de Thermo Fisher Scientific Oy.
Thermo Fisher Scientific Oy